

## CONSERVATIVE



### LIQUID DAM

#### Features

Defendo is a gel that is used to protect the gums or other dental surfaces before bleaching operations, and other procedures requiring protection intraoral.

The gel of fluid consistency guaranteeing exact positioning of the product and a great sealing of interproximal spaces.

Thanks to the intense blue color, the use is simple and well controllable.

Through the applicator tip (flow tip) to insert on the syringe, Defendo can be applied with punctiform precision even in areas that are difficult to reach. The product does not drip when applied to the surface to be protected. The product does not become rigid after the polymerization, but is flexible and can be easily removed in one piece without tearing.

#### Indications

- Gingival protection of teeth subjected to whitening treatment.
- Isolation of the surgery field, around small III or V class cavities, preventing contamination during the restoration.
- Repair of lacerations, or completion the closure of dams in latex.
- To eliminate undercuts, before taking impression, holding precision strike, plants and interproximal niches.
- Allow mounting and maintenance in position of the mylar strips used as matrix.

- Coating the surfaces adjacent to areas to be etched with hydrofluoric acid during the direct repair of fractured ceramic crowns

#### Instructions for use

Remove the cap from the syringe with counterclockwise rotation movement and replace it with the special metal applicator tip.

After using the product, re-plug immediately the protective cap.

Make sure the surface to be treated is cleansed adequately.

Apply a layer of about 1-2 mm on surfaces to be protected before the operations of bleaching, etching and other processes requiring protection intraoral or undercuts, to be eliminated before taking the footprint.

Light cure for about 10 seconds until the Dam proves hard (the time may vary slightly depending on the type of lamp used).

At the end of the treatment, remove the dam with the aid of a common dental probe, and then ask the patient to perform an abundant rinsing of the mouth.

#### Composition

Gel-base dimetacrilic aliphatic and aromatic monomers

#### Packaging

N° 3 Syringes 3g.  
N° 6 Flow tips  
Leaflet

Medical device class I

