

## CONSERVATIVE

# ipoclor<sup>5</sup>

### The excellent solutions for cleaning root canal

A choice based on scientific evidence IPOCLOR 5 Sodium hypochlorite at 5% active chlorine Ready to use solution based on sodium hypochlorite at 5% for the irrigation of root canals during reaming operations.

The high quality of glass packaging ensures greater stability of the product over time.

### Characteristics

High capacity of dissolving the organic component because the high concentration (5.25) and the high pH;  
High antimicrobial activities also for Enterococcus faecalis;

Medical device IIa for the exclusive dentist use CE 0426

Do not mix with acids because it would release toxic gases.

Do not inject - do not ingest

Keep out of the reach of children.

### Packaging

Multidose bottle 250 ml

### Composition

Sodium hypochlorite at 5% active chlorine

### Instructions for use

With a disposable syringe with blunt canal washing needle, aspirate the solution from the bottle and to wash canal.

Medical device IIa  0426

